


# TOP NEUROPOLE SC NEEDLES

All-in-one disposable thermocouple needles

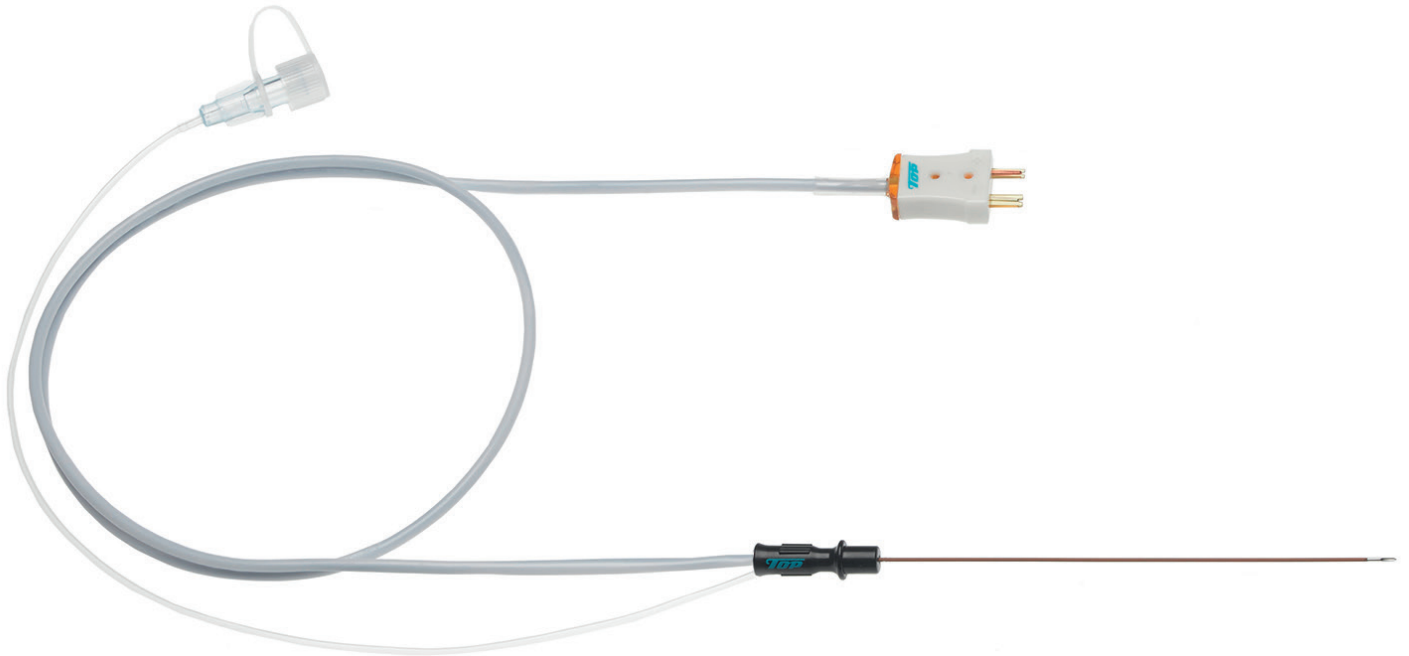

## Insert, Stimulate, Inject and Lesion with Accuracy

- No dislocation due to movement of thermocouples
- Intermediate cables for most brands available
- The most economical option on the market
- Always perfectly accurate
- No cross contamination

### Top NeuroPole SC Needles for Thermolesion and PRF Exposure

Imagine an RF procedure without the use of a separate thermocouple. An RF procedure without risks of dislocation due to thermocouple insertion and removal. Top makes it possible with the SC Neuropole needles. Typically designed for temperature-controlled radiofrequency lesioning, the Top Neuropole SC needles will prove to be a great enhancement in your routine. Available in a variety of lengths and gauge-diameters, depending on your needs.

#### Optimising the Procedure

The Top Neuropole SC needles are equipped with a built-in thermocouple temperature sensor and an additional tube with Luer lock, to effectively optimise the thermolesion procedure.

Exchanging thermocouple and syringe, and all involved operational steps, will no longer be necessary after localising a nerve's proper position. Any risk of dislocation during removal and re-insertion of the thermocouple or during connection, application and removal of the syringe is therefore eliminated.

Insulated with a high grade medical Teflon-coating, the needles guarantee a safe, reliable procedure and an ever perfect accuracy.

#### Eliminating Cross Contamination Issues

The Top Neuropole SC needles are disposable cannulas, intended for single use. They are individually packed in an easy to use, sealed medical pouch. Eliminate sterilisation logistics and costs. The Top Neuropole SC needles are the most economical option on the market.

#### Compatibility

The Top Neuropole SC needles are adequately designed for all kinds of RF interventions. Intermediate cables for all major RF generator brands are available.

#### Available Sizes

Depending on your preferences, the Neuropole SC needles are available with various features, in a diverse range of lengths and diameters.



### NeuroPole SC - Ordering Information

TOP NEUROPOLE SC-K, disposable thermocouple needles				
Order number	Needle length	Gauge	Active tip	Package
627124	60 mm	22 G	5 mm	10 pcs/box
627106	60 mm	21 G	10 mm	
E63281	100 mm	22 G	2 mm	
627125	100 mm	22 G	5 mm	
627107	100 mm	21 G	10 mm	
E63286	100 mm	18 G	10 mm	
627126	150 mm	20 G	5 mm	
627105	150 mm	20 G	10 mm	
627114	200 mm	23 G	5 mm	

TOP NEUROPOLE SCU, disposable thermocouple needles, echogenic				
Order number	Needle length	Gauge	Active tip	Package
627148	60 mm	21 G	5 mm	10 pcs/box
627147	100 mm	21 G	5 mm	

TOP NEUROPOLE SCC, disposable thermocouple needles, curved				
Order number	Needle length	Gauge	Active tip	Package
627261	60 mm	21 G	10 mm	10 pcs/box
627262	100 mm	20 G	10 mm	
627263	150 mm	20 G	10 mm	

TOP NEUROPOLE SCCBL, disposable thermocouple needles, curved, blunt tip				
Order number	Needle length	Gauge	Active tip	Package
E63284	100 mm	20 G	10 mm	10 pcs/box
E63285	145 mm	20 G	10 mm	

## Intermediate Cables - Ordering Information

Intermediate SC cables for the following (P)RF generator brands are available:		
Order number	Description	Package
EM5517B	Intermediate (P)RF cable to Baylis	1 pc/pouch
EM5517BMRF	Intermediate (P)RF cable to Baylis Generator MRF	
EM5517C	Intermediate (P)RF cable to Cosman	
EM5517DM	Intermediate (P)RF cable to Diros Multichannel	
EM5517N	Intermediate (P)RF cable to Neurotherm	
EM5517S	Intermediate (P)RF cable to Stryker	
EM5517TM	Intermediate (P)RF cable to Thermediko Generator NK1	

### The Key to Medical Innovation

Innovation in the fields of Anesthesiology and Pain Management is at the heart of Equip Medikey. Joined by true pioneers, we develop new products with unique features, for safer anesthesia and effective pain treatments. Valuable solutions for many practitioners and patients, all over the world.



Equip Medikey BV / Edisonstraat 16J / 2809 PB Gouda / The Netherlands

T +31 182 57 32 93 / info@equip.nl / www.equip.nl

Rabobank Utrecht / IBAN NL05 RABO 0158 4999 05 / BIC/SWIFT RABONL2U / KvK 24400829 / VAT NL817000240B01

